




Cannula

Nasal cannula are clear, latex free, oxygen delivery devices which are used for oxygen flow rates under 6 LPM. Cannula come in a variety of sizes. The prongs can be curved or straight pending on manufacturer and model.

NOTE: Flow rates above 4 LPM are not well tolerated.

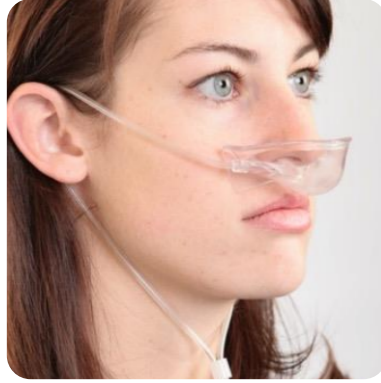

Bi-Flow Nasal Cup

Bi-Flow Nasal Cups are an excellent alternative to nasal cannula for those with irritated nasal passages. The soft pliable cup sits over the nose encircling it in a pool of oxygen from which the patient breathes.

Bi-flow nasal masks are suitable for oxygen flow rate of up to 3 LPM and may be used with a humidifier.



Masks

Simple oxygen masks are clear, latex free oxygen delivery devices used for oxygen flow rates over 5 LPM. Simple oxygen masks come in a variety of sizes and can fit under the mouth or chin depending on manufacturer and model.

Oxymisers

The oxymiser is an oxygen conserving device. Oxygen is stored in the 20 ml reservoir and inhaled only during the inhalation part of the breathing cycle via a nasal cannula.

An oxymiser requires as much as 75% less oxygen flow rate to deliver the same fractional inspired concentration and therefore must be prescribed by a health professional to ensure that oxygen delivery is precise.

They are suitable for oxygen flow rate of up to 5 to 12 LPM and cannot be used with a humidifier.

Oxymisers cannot be used with OCD technology or humidifiers

NOTE: THIS PRODUCT (OXYMISER) MUST BE PRESCRIBED BY A HEALTH PROFESSIONAL. THE GENERAL PUBLIC CANNOT PURCHASE THIS PRODUCT WITHOUT A PRESCRIPTION.



Firesafe™ valves

Firesafe™ valves can be connected to medical oxygen delivery tubing connecting your accessories to provide an extra line of defense. Valves limit the impact of an oxygen fire by automatically arresting the oxygen flow in the event of a fire in the downstream delivery tubing.



Comfort Solutions

We also provide a selection of products to improve the comfort of accessories used for oxygen therapy.

NozOil™

moisturises nasal tissues to prevent dryness and crusting.



Comfy Ears™, EZ Wraps™

are cannula padding solutions which aim to reduce skin irritation and chafing.



Connectors

Connectors are used to connect oxygen tubing and/or oxygen cannula to extension tubing.

Features:

Leak-free coupling and silky smooth 360 rotation.



Extension Tubing

Extension tubing can be added to a nasal cannula or mask to add length without affecting the flow of oxygen from a stationary concentrator. A maximum of 50 feet (15 metres) is available.



Humidifiers

Humidifiers can provide around 20% humidity in the inspired gases. It does so by bubbling the oxygen through water, increasing the oxygen's moisture content.

They are ideal for use with all stationary oxygen concentrators and cylinders.



Water Traps

When a humidifier is used, condensation may accumulate inside the tubing. The use of a water trap can help resolve this problem.



Item Code	Product Description
Cannula	
6437S	Adult, SOFT Tubing - 1.2M
6438S	Adult, SOFT Tubing - 2.1M
5226S	Adult, with 30cm supply tube
359S	Adult, without supply tube
5222S	Adult, without supply tube, with EZ Wraps
121S	Infant - 2.1M
122S	Interm infant (3-12mth) - 2.1M
124S	Pediatric - 2.1M
125S	Premature - 2.1M
Bi-Flow Nasal Cup	
1394S	Adult, Bi-Flow Nasal Cup - 2.1M
Masks	
152S	Adult, Medium Concentration with Tubing - 2.1M
154S	Adult, Medium Concentration without Tubing
158S	Adult, High Concentration with Part Rebreather Vent & Tubing - 2.1M
161S	Adult, High Concentration with Non-Rebreather Vent & Tubing - 2.1M
936S	Pediatric, Medium Concentration with Tubing - 2.1M
157S	Pediatric, Medium Concentration without Tubing
205S	Pediatric, High Concentration with Non-Rebreather Vent & Tubing - 2.1M
206S	Pediatric, High Concentration with Non-Rebreather Vent & Tubing - 2.1M
Oximiser	
230S	Nasal Pendant
Comfort Solutions	
3939S	Comfy Ears
2983S	EZ Wraps
3455S	NozOil - 15ML
Extension Tubing	
5227S	10CM
843S	2.1M
1975S	4.6M
4179S	10.7M (green)
1977S	15M
Accessories	
6518S	Firesafe Valve
4870S	Flow Meter
1676S	Oxygen Tubing Swivel Connector (Low Flow)
5632S	Y-Piece Connector (High Flow)
146S	Humidifier
2800S	Humidifier Tubing - INVACARE ONLY (part comes included with the Philips Everflo)
1793S	Water trap

PLEASE NOTE

- This list is not a full listing of our product range, it is a summary of the most frequently purchased products.
- ALWAYS READ THE LABEL OF EACH PRODUCT AND FOLLOW THE DIRECTIONS FOR USE