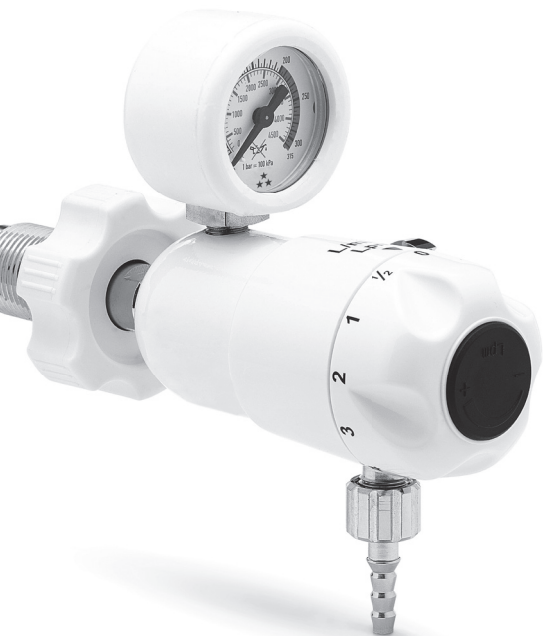
CEODEUX
MEDITEC

ALPINOX REGULATOR

Key							
View A	View B	View C	View F	View G			
2	Flowmeter control wheel	3	Medical connector pressure outlet	6	"Direct drive" manometer	Fixed pressure regulator with ball rotameter 0 - 15 l/min O2 ref. 9. These devices will be fitted with a removable nipple assembly, view G.	Removable nipple assembly option: - No.10: O-ring - No.11: Hose nipple - No.12: Screw nut in compliance with national standards
4	Bourdon gauge tube manometer	1	Flowrate outlet	7	Bottle bracket connector		
5	Bottle connector						

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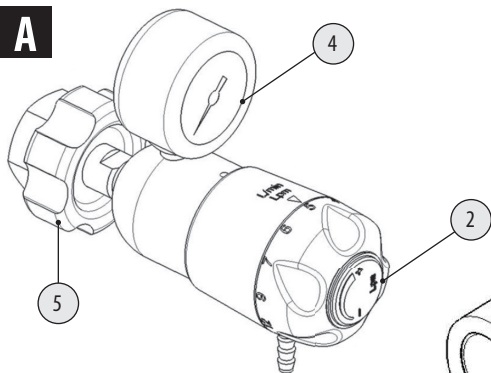
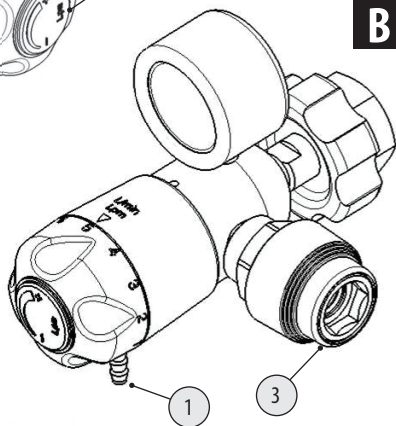
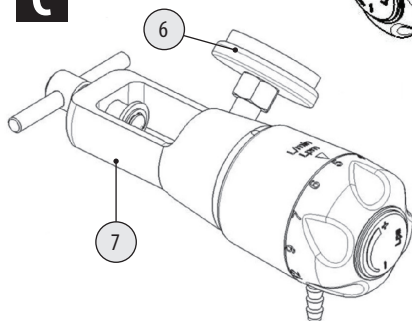
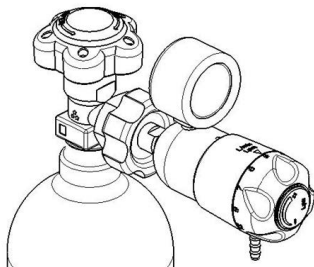
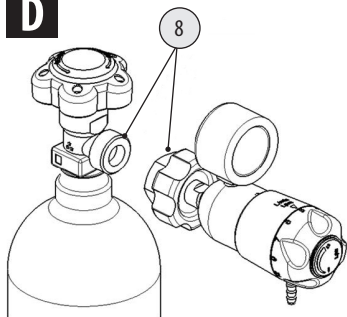
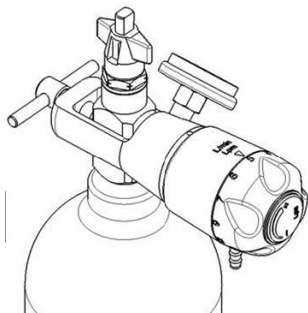
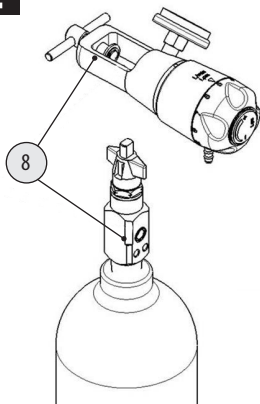
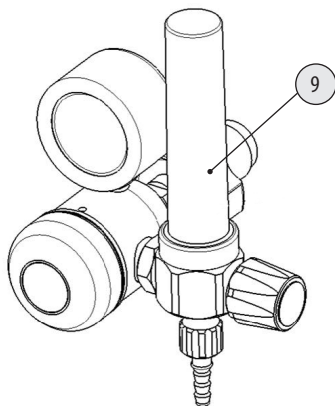
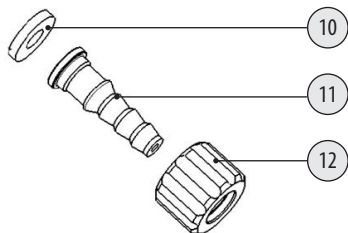
F**G**

ALPINOX

ALPINOX REGULATORS ARE MEDICAL PRESSURE REGULATORS, FLOW METERS FOR OXYGEN THERAPY IN HOSPITALS, FOR HOME CARE AND IN DEAMBULATION.

REGULATOR NAME: ALPINOX

High-pressure regulator for standardised fixed regulation, with cylinder or pin-index flow meter. Optionally, this regulator can be fitted with a low-pressure medical connector.

A) Mounting the regulator on the gas bottle

NOTE: Carefully place the bottle so as to avoid any risk of it falling, and secure it in place if possible. Before connecting the regulator, check that the high-pressure valve connector is free of dust or foreign bodies (see figures D and E ref. 8).

ATTENTION: Make sure that hands are grease-free when manipulating the regulator and the gas valve.

- 1) Remove the bottle valve protection.
- 2) Remove the high-pressure connector protection stopper from the regulator. Store it away in a clean location.
- 3) Ensure that the bottle valve and the regulator connector are clean (no dust or foreign bodies).
- 4) Check that the O-ring on the regulator inlet connector is in good condition, and is not scratched, torn or creased. The O-ring must always be maintained in perfect position.
- 5) Hand-tighten the regulator inlet connector in the bottle valve so as to obtain a sealed connection (tightening using a tool is strictly prohibited for O-ring seals).

NOTE: If a seal gasket is faulty, maintenance must be carried out by a certified technician in accordance with the instructions provided in the Ceodeux SA maintenance documentation. Only use original parts provided by Ceodeux SA.

6) Connecting to the user device

- either to the oxygen therapy cylinder nipple outlet
- or to the pressure outlet (medical connector in compliance with country specific standards)

ATTENTION: The flow meters must be at position 0 when connecting to the bottle and when slowly opening the high-pressure valve of the bottle. The cylinder outlet must never be used for supplying machines or other devices.

B) Using the regulator.

- 1) In accordance with recommendations for oxygen application, open the bottle valve slowly. You are recommended not to stand opposite the regulator or high-pressure gas flow. Check the quantity of gas remaining in the bottle by means of the regulator manometer.

IMPORTANT: In the event of a leak between the regulator and the bottle close the bottle valve immediately and wait for the leak to stop. Open the chamber outlet to ensure that there is no remaining pressure in the regulator. Under no circumstances should the connector be unscrewed or retightened if the manometer indicates a positive pressure. Remove the regulator from the bottle and call a Ceodeux SA certified technician to carry out maintenance.

- 2) Adjust the desired gas flow using the flowrate selector switch. Do not try to place the selector switch between the flowrates indicated on the control wheel. Do not use simultaneously the pressure outlet and the flow outlet.
- 3) After use, close the bottle valve then:
 - Disconnect the cylinder hose and wait for the leak to stop.
 - Ensure that the pressure reading on the manometer is 0.
 - Set the cylinder to position 0.

If you were using the pressure outlet:

- Disconnect the hose from the medical connector.
- Set the cylinder to max. position and wait for the leak to stop.
- Ensure that the pressure reading on the manometer is 0.
- Set the cylinder to position 0.

C) Disconnecting the bottle from the regulator.

- 1) Unscrew by hand the regulator inlet connector (do not use any tool and do not force).
- 2) Store the regulator in a dry and clean location after replacing the protective stopper on the inlet connector.

D) Cleaning and disinfection.

The Alpinox regulator should only be cleaned with a damp cloth. Disinfect with an alcohol-based solution (CE).

ATTENTION: It is strictly prohibited to use products containing ammonia or triethanolamine in whatever concentrations.

E) Maintenance

Product is designed for 10 years lifetime from manufacturing date under normal conditions of use as mentioned in the instruction for use. No preventive maintenance is requested during lifetime under normal conditions of use. Regular controls are required. Frequency depends on intensity of use

but must not exceed 5 years. CEODEUX S.A. recommends annual controls (visual inspection, operational check, checks for leaks and of cleanliness). In case of failure or damage, repairs are to be performed by CEODEUX S.A. or by certified and trained agents and with spare parts from the original manufacturer, CEODEUX S.A.

F) Safety conditions

- Comply with safety instructions linked to the use of gases.
- Do not place gas bottles near a source of heat or close to inflammable materials.
- Do not smoke in proximity to the device.
- No oily/greasy product (oil, gasoline, butter, ointment, etc.) should come into contact with the device.
- Check that the bottle is properly secured so that there is no risk of it falling.

Symbol:

CE 0029 in compliance with European Directive 93/42/EEC of June 14 1993 amended 2007/47/CE of September 5 2007 on medical devices, established by notified body 0029

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